



CLOSURE RESEALS AFTER EACH DOSE!

- Increase Child Safety
- Help Prevent Contamination
- Avoid Accidental Spills
- Use with Original C/R Caps
- Improve Dosing Accuracy




Patented SealSafe™ Adapters allow oral syringes to draw the exact amount needed while reducing the risk of spill, evaporation and contamination. They fit snugly into most bottles in their size range allowing original child-resistant (C/R) caps to fit over the closures. To use, simply insert the tip of an oral syringe into a SealSafe™ Adapter seal, draw a measured dosage and remove the tip. The adapter quickly recloses providing a safe and clean seal.

Medications are less accessible when the child-resistant cap is removed.

Bottle Size Selection:

SealSafe™ Adapters feature flexible sealing rings for compatibility with a range of bottle sizes. Catalog #760000 (size 24) fits snugly into most bottles with necks in its I.D. (inner diameter) range of 15.5mm - 18.25mm.

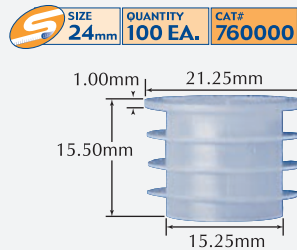

Bottles should be selected for smooth inside necks sized within the stated inner diameter range. A smooth neck optimizes the ability of the closure to provide an effective seal.

Using SealSafe™ Adapters:



1. Select a bottle with a compatible neck size.



2. Remove the screw-cap.



3. Press a SealSafe™ Adapter into the neck of the bottle.



4. Insert the tip of an oral syringe into the seal, turn the bottle upside-down and draw a desired dosage.



5. Turn the bottle right-side-up and remove the syringe tip. The SealSafe™ Adapter reseals after the tip is removed. Replace the cap onto the bottle.

! SealSafe™ Adapters should be tested for use with specific containers, liquids and oral syringe tips to confirm fit and compatibility and to ensure optimal performance.

Ordering Information:

Cat #	Description
760000	SealSafe™ Closure, 100/Box
760009	SealSafe™ Starter Kit*

SealSafe™ Closure Pricing: Distributor & volume discounts are available. Contact Andwin for samples and pricing today!

*Kit includes 20 each of SealSafe™ Closures, Amber PET Bottles & C/R Caps.



Patented SealSafe™ Closures make liquid medication storage and dispensing safer and easier than ever!

SealSafe™ is a Trademark of The Andwin Corporation.
 Penetrex® is a Registered Trademark of The Andwin Corporation.
 Self-Sealing Adapters are Protected by Patents and Patents Pending.